



reducecept

# Reducecept

## Dashboard manual

**Reducecept**

Blokhuisplein 40, 8911 LJ Leeuwarden, Nederland

E-mail: [info@reducecept.com](mailto:info@reducecept.com)

Tel: +31 85 13 08 260

CoC: 74239678

VAT: NL859821432B01

[www.reducecept.com](http://www.reducecept.com)

Reducecept is part of RelieVR B.V.



# Dashboard

The Reducept dashboard has been specifically developed for therapists. The dashboard enables you to manage multiple users and device, and to monitor the performance of the players you are treating.

## Gaining access to the dashboard

1. You have received an e-mail from Reducept which contains an invitation to create a dashboard account. Click on the link to go to the dashboard.

2. Choose a password and confirm it on the log in screen.

reducept Log in Manual English Nederlands

reducept

Create password

E-mail demo@example.net

Password

Confirm password

Your password needs to contain at least 1 uppercase character, 1 lowercase character and 1 number.


Set password

Reducept - dashboard is offered by RelieVR BV. Privacy policy - Manual

© 2017 - 2019 and powered by Grendel Games. All Rights Reserved. Gentsele - Bootstrap Admin Template by Colorlib




3. You will now see the Home-screen of the dashboard which shows a summary of the activities of your players within the last month.

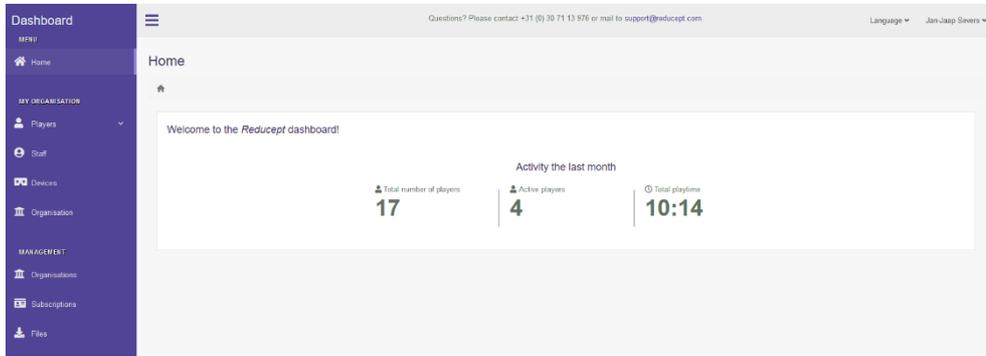




## How to create new players

1. Open the 'Players' menu which is located below 'My organisation'.

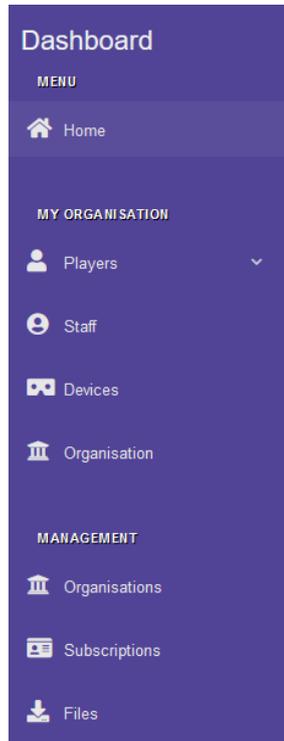




2. Click on the 'Players' menu-option. You will now see an overview of all players who are linked to your organisation.

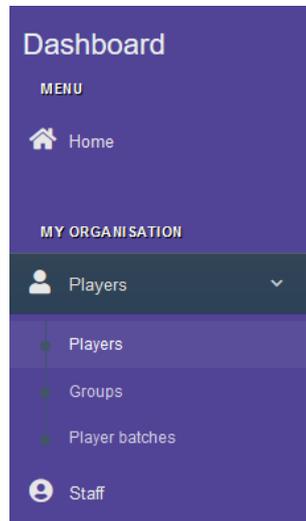

3. Click the 'Add player' button. Several empty input fields will appear in the player overview.

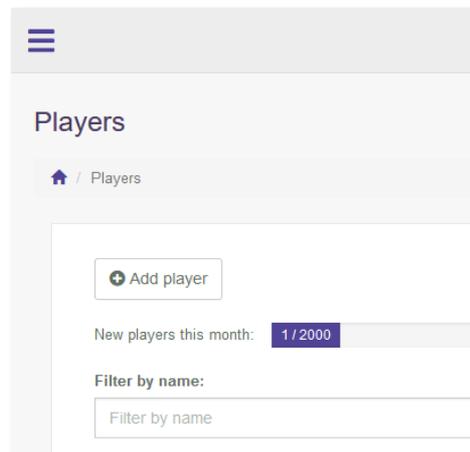



4. Fill in the new player's data, and then click 'Save'.

First name	Last name	Group	Active until	Actions
<input type="text" value="John"/>	<input type="text" value="Doe"/>	<input type="text" value="None"/>	<input type="text" value="19-03-2020"/>	<input type="button" value="Save"/> <input type="button" value="Cancel"/>

5. A new screen will appear in which more detailed player data can be stored, and where you can store notes. Click on 'Save'.

Profile

Home / Players / John Doe / Profile

Here you can edit more detailed information about this player.

Name John Doe

E-mail

Date of birth

Gender Unknown

Notes



6. The new player has been added and will appear in the player overview screen.

Players

🏠 Players

[Add player](#)

New players this month: **2 / 2000**

Filter by name:


Filter by group:

First name	Last name	Group	Active until	Actions
Grendel	Test #2	None	09-04-2020	<input type="button" value="View"/> <input type="button" value="Edit"/> <input type="button" value="Actions ▾"/>
Grendel	Test #4	None	03-04-2020	<input type="button" value="View"/> <input type="button" value="Edit"/> <input type="button" value="Actions ▾"/>
John	Doe	None	19-03-2020	<input type="button" value="View"/> <input type="button" value="Edit"/> <input type="button" value="Actions ▾"/>




## How to look at player statistics

1. Open the 'Players' menu which is located below 'My organisation'.

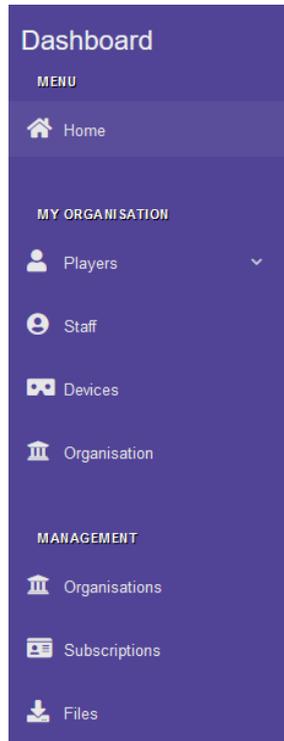



2. Click on the 'Players' menu-option. You will now see an overview of all players who are linked to your organisation.

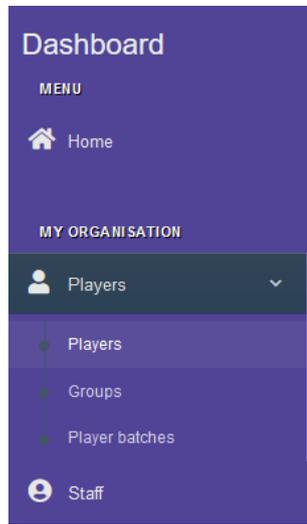

3. Click the 'View' button next to the player whose statistics you want to see.

John	Doe	None	19-03-2020	<a href="#">View</a> <a href="#">Edit</a> <a href="#">Actions</a>
------	-----	------	------------	---



- The statistics screen shows a summary of all sessions this player has completed within a selected period of time.

The screenshot shows the Reducept interface for a player named John Doe. At the top, there is a navigation bar with a menu icon, contact information, and language/user settings. Below this, the player's name 'John Doe' is displayed, along with a breadcrumb trail 'Players / John Doe'. A 'Period selection' section contains two date pickers: 'From' (06-12-2019) and 'To' (20-12-2019), with a 'View' button below them. The main content area is divided into two columns. The left column, titled 'Pain locations', features a 3D human figure with a red heatmap overlay. The right column, titled 'Summary', contains a paragraph of text and a 'Statistics' section. The statistics are presented in a table-like format with four columns: 'Number of sessions' (0), 'Total playtime' (0:00), 'Average start pain score' (0), and 'Average end pain score' (0).

- You can see statistics from a specific period by changing the start and end date of the 'Period selection' section. Then click 'View'. The statistics screen will then show player statistics from the desired time period.


This is a close-up view of the 'Period selection' section from the previous screenshot. It shows two date input fields: 'From' with the value '06-12-2019' and 'To' with the value '20-12-2019'. Each field has a calendar icon to its right. Below the date fields is a blue button with a magnifying glass icon and the text 'View'.



## How to link devices to your organisation

1. To link Oculus headsets to your organisation you log into the Reducept app with your dashboard account. To do so, follow the steps in the 'Log into the Reducept app' chapter of the Reducept Oculus Go manual.

2. Open the 'Devices' menu which is located below 'My organisation'.

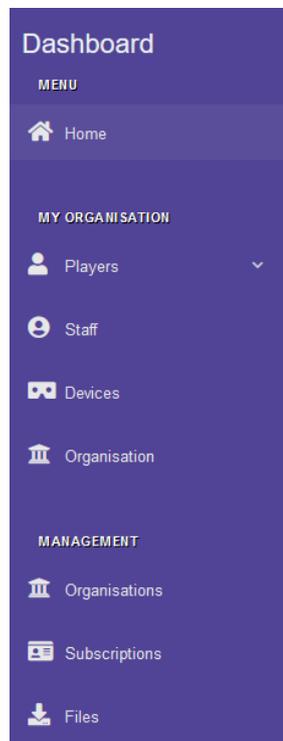



3. You now see the device overview screen. The new device has been added to this overview.

The screenshot shows the Reducept dashboard with the 'Devices' section active. The table below lists the devices and their current status.

#	Name	Logged in player	Actions
1	Oculus Go	None	Log in, Edit, Delete
2	Oculus GO	None	Log in, Edit, Delete
3	Oculus Go	None	Log in, Edit, Delete
4	Oculus Go	None	Log in, Edit, Delete
5	Galaxy Tab S2	None	Log in, Edit, Delete
6	DESKTOP:7MLS46L	Grendel Test #1	Log out, Edit, Delete
7	Oculus Go	None	Log in, Edit, Delete
8	Oculus Grendel	Grendel Test #7	Log out, Edit, Delete
9	New Device	None	Log in, Edit, Delete
10	Android Phone	Grendel Test #4	Log out, Edit, Delete


At the bottom of the table, there is a pagination control showing '1 2 3' with arrows on either side.

Footer text: Reducept - dashboard is offered by ReleVR BV. Privacy policy - Manual. © 2017 - 2019 and powered by Grendel Games. All Rights Reserved. Gendels - Bootstrap Admin Template by Colorlib



## How to log in and log out player on the Oculus Go

4. Open the 'Devices' menu which is located below 'My organisation'. You now see the devices overview screen.

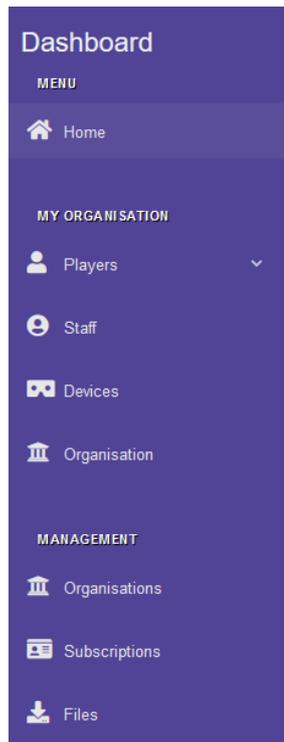



5. Click on the 'Log in' button next to the device you want to use.

Dashboard

Questions? Please contact +31 (0) 30 71 13 976 or mail to support@reducept.com

Language Jan-Jaap Severs

### Devices

Devices

To play Reducept on a mobile device, that device needs to be linked to your license. On this screen you can manage your devices and log in a player account on a device.

+ Add device

#	Name	Logged in player	Actions
1	Oculus Go	None	Log in Edit Delete
2	Oculus GO	None	Log in Edit Delete
3	Oculus Go	None	Log in Edit Delete
4	Oculus Go	None	Log in Edit Delete
5	Galaxy Tab S2	None	Log in Edit Delete
6	DESKTOP-TMS48L	Grendel Test #1	Log out Edit Delete
7	Oculus Go	None	Log in Edit Delete
8	Oculus Grendel	Grendel Test #7	Log out Edit Delete
9	New Device	None	Log in Edit Delete
10	Android Phone	Grendel Test #4	Log out Edit Delete

Reducept - dashboard is offered by RelivVR BV  
Privacy policy - Manual

© 2017 - 2019 and powered by Grendel Games. All Rights Reserved.  
Grendel - Bootstrap Admin Template by Grendel

6. Enter (part of) the player name and click on 'Assign'.

Dashboard

Questions? Please contact +31 (0) 30 71 13 976 or mail to support@reducept.com

Language Jan-Jaap Severs

### Select player

Devices | Oculus Go | Select player

Select a player that is going to use this device by entering (a part of) their name here.

Player John Doe

Assign

Reducept - dashboard is offered by RelivVR BV  
Privacy policy - Manual

© 2017 - 2019 and powered by Grendel Games. All Rights Reserved.  
Grendel - Bootstrap Admin Template by Grendel



7. The selected player is now linked to the device.

Player successfully logged in!

To play Reducept on a mobile device, that device needs to be linked to your license. On this screen you can manage your devices and log in a player account on a device.

+ Add device

#	Name	Logged in player	Actions
1	Oculus Go	John Doe	Log out Edit Delete
2	Oculus GO	None	Log in Edit Delete
3	Oculus Go	None	Log in Edit Delete
4	Oculus Go	None	Log in Edit Delete
5	Galaxy Tab S2	None	Log in Edit Delete
6	DESKTOP-7ML54RL	Grendel Test #1	Log out Edit Delete
7	Oculus Go	None	Log in Edit Delete
8	Oculus Grendel	Grendel Test #7	Log out Edit Delete
9	New Device	None	Log in Edit Delete
10	Android Phone	Grendel Test #4	Log out Edit Delete


Reducept - dashboard is offered by RalisVR BV.  
Privacy policy - Manual

© 2017 - 2018 and governed by GrendelGames. All Rights Reserved.  
GrendelG - Bootstrap Admin Template by Colorlib



## Allowing home usage

1. Click the 'Players' button which is located below 'My organisation'. You now see the player overview screen.

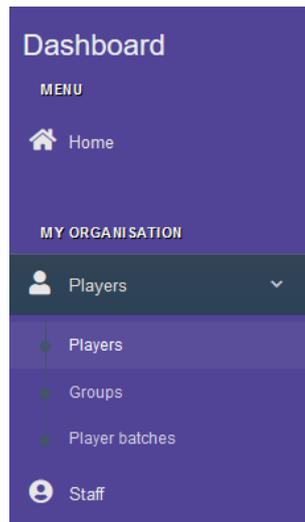

2. Click on the 'Actions' button, followed by the 'Manage home usage' button next to the right player.

The screenshot shows the 'Players' overview screen. At the top, there's a header with 'Players' and a sub-header 'Players'. Below this, there's a table with columns: 'First name', 'Last name', 'Group', 'Active until', and 'Actions'. The table contains 10 rows of player data. The 'Actions' column for the second row (Grendel, Test #2) is expanded, showing options like 'View', 'Edit', 'Actions', 'Edit profile', and 'Manage home usage'. The 'Manage home usage' option is highlighted.

First name	Last name	Group	Active until	Actions
John	Doe	None	15-03-2020	View Edit Actions
Grendel	Test #2	None	09-04-2020	View Edit Actions Edit profile Manage home usage
Grendel	Test #4	None	03-04-2020	View Edit Actions Create
Joop	Test2	None	17-03-2020	View Edit Actions
Grendel	Test #7	None	01-03-2020	View Edit Actions
Grendel	Test #3	None	29-02-2020	View Edit Actions
Ewin	test 1	None	21-02-2020	View Edit Actions
Test	test	None	21-02-2020	View Edit Actions
Grendel	Test #10	None	27-02-2020	View Edit Actions
Grendel	Test #1	None	01-01-2020	View Edit Actions



3. Click on the 'Allow home usage' button. The player will receive an e-mail with instructions for creating a Reducept account for home usage.

Dashboard

Questions? Please contact +31 (0) 30 71 13 976 or mail to support@reducept.com

Language Jan-Jaap Severs

### Players

Players

Add player

New players this month: 2 / 2000

Filter by name: Filter by name

Filter by group: All groups

First name	Last name	Group	Active until	Actions
John	Doe	None	19-03-2020	View Edit Actions -
Grendel	Test #2	None	09-04-2020	View Edit Manage home usage
Grendel	Test #4	None	03-04-2020	View Edit Delete
Joep	Test2	None	17-03-2020	View Edit Actions -
Grendel	Test #7	None	01-03-2020	View Edit Actions -
Grendel	Test #3	None	29-02-2020	View Edit Actions -
Erwin	Test 1	None	27-02-2020	View Edit Actions -
Test	test	None	27-02-2020	View Edit Actions -
Grendel	Test #10	None	27-02-2020	View Edit Actions -
Grendel	Test #1	None	03-01-2020	View Edit Actions -